VetPro[®] 1000 Mobile Dental Delivery System

[English](#)
[Español](#)
[Français](#)

For Models:
8000-001
8000-002
8000-003
8000-004
8000-005
8000-006
8000-006S
8000-007
8000-007S
8000-008
8000-008S

User's Guide

Product Information

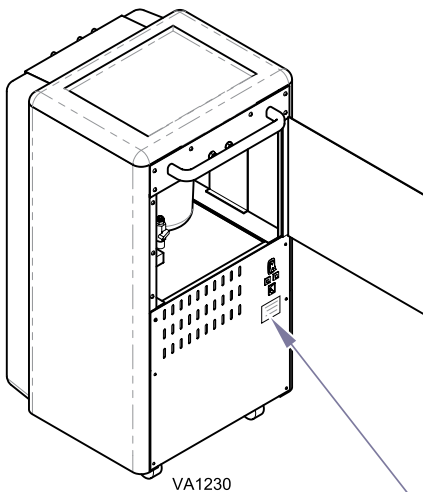
(The information below is required when calling for service.)

Dealer :

Date of Purchase:

Model / Serial Number:

**Midmark Authorized Service
Company:**

Model / Serial Number Location

Table of Contents

Important Information

Safety Symbols	4
Intended Use	5
Electromagnetic Interference	5
Disposal of Equipment	5
Authorized EU Representative	5
Transportation / Storage / Operating Conditions	5

Transport

Transporting Dental System	6
--------------------------------------------------	---

Operation

Electrical System	7
Water System	8
Air System	9
Delivery System	10
Handpiece Care	11
Syringe Tip Replacement & Handpiece Installation	12
Drive Air Adjustment & Locking Casters	13
Air Pressure Regulator & Water Flow Adjustment	14
Coolant Air Adjustment	15

Cleaning / Maintenance

Cleaning	16
Maintenance	18
Calling for Service	22

Specifications

Drawing	23
Weights / Dimensions	24
Electrical	25
Model Identification / Compliance Chart	26

Warranty

Warranty Information	27
--------------------------------------------	----

For installation information go to:

[Technical Library at Midmark.com](#)

Important Information

Safety Symbols

WARNING

Indicates a potentially hazardous situation which could result in serious injury.

Caution

Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices.

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage if not avoided.

Note

Amplifies a procedure, practice, or condition.

Proper Shipping Orientation

Handle With Care

Type B, Applied Part

Maximum stacking height (Do not stack)

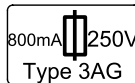
Humidity Limit

Type BF, Applied Part

Fragile

Fuse Rating Specifications

Protective Earth Ground

Keep Dry

Pressure Limit

LED Scaler Handpiece Duty Cycle

Refer to Instruction Manual/Booklet

Temperature Limit

TUV - Domestic Models Only

Do Not Tumble

Intended Use

The Midmark® VetPro® 1000 is a mobile dental delivery system intended to give qualified veterinarians and/or technicians a self-contained, compact, portable unit for performing veterinary dental procedures.

Electromagnetic Interference

This product is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this cart:

- Remove interfering device from room
- Plug cart into an isolated circuit
- Increase separation between Midmark product and interfering device
- Contact Midmark if interference persists

Disposal of Equipment

At the end of product life, the unit(s), accessories, and other consumable goods may be contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment and other consumable goods.

Authorized CE Representative

Countries in the EEC should direct all questions, incidents, and complaints to Midmark’s Authorized CE representative listed below:

Midmark EMEA Ltd
 Beech House
 First Floor, East Wing
 Ancells Business Park
 Fleet
 Hampshire GU51 2UN
 United Kingdom
 Tel: + 44 (0) 1252 360 940
 Fax: + 44 (0) 1252 360 941

Transportation / Storage / Operating Conditions

Transportation / Storage Temperature Range:23°F to 100°F (-5°C to 38°C)
 Operating Temperature Range:50°F to 104°F (+10°C to +40°C)
 Transportation / Storage Relative Humidity:10% to 90% (non-condensing)
 Operating Relative Humidity:30% to 75% (non-condensing)
 Transportation / Storage Atmospheric Pressure:50 kPA (7.2 psi) to 106 kPA (15.3 psi)
 Operating Atmospheric Pressure:700 hPA (20 in Hg) to 1060 hPA (31 in Hg)

Transport

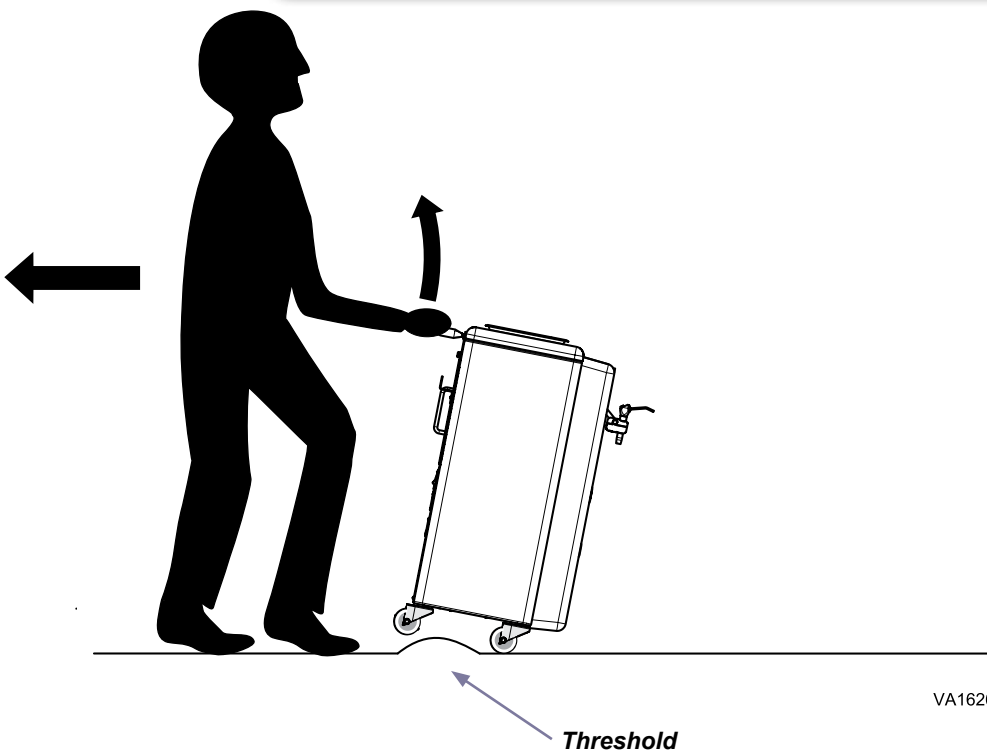
WARNING

Exercise care when transporting over thresholds and other low lying objects. Failure to do so can lead to injury.

When transporting over large thresholds or steps...

A) Pull cart up and over onto level surface.

Operation

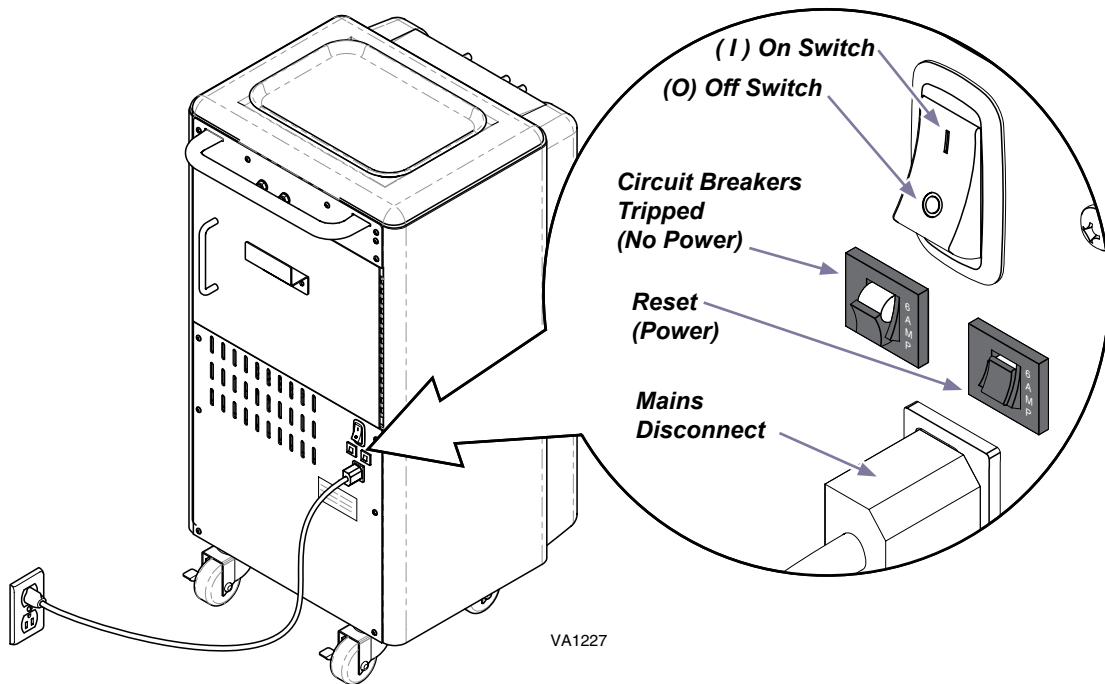
Electrical System

On/Off Switch

On/Off Switch is located on the back of the cart. It controls the main power source and must be ON to operate the delivery system.

Circuit Breakers

There are two circuit breakers located on the lower back panel of the cart. If the unit's maximum load is exceeded, the circuit breakers interrupt power to the unit. Maximum load is 115 VAC, 6 amps (domestic) 230 VAC, 3 amps (export).

Mains Disconnect

Power can be terminated by unplugging the mains disconnect or turning switch off.

Operation

Water System

Caution

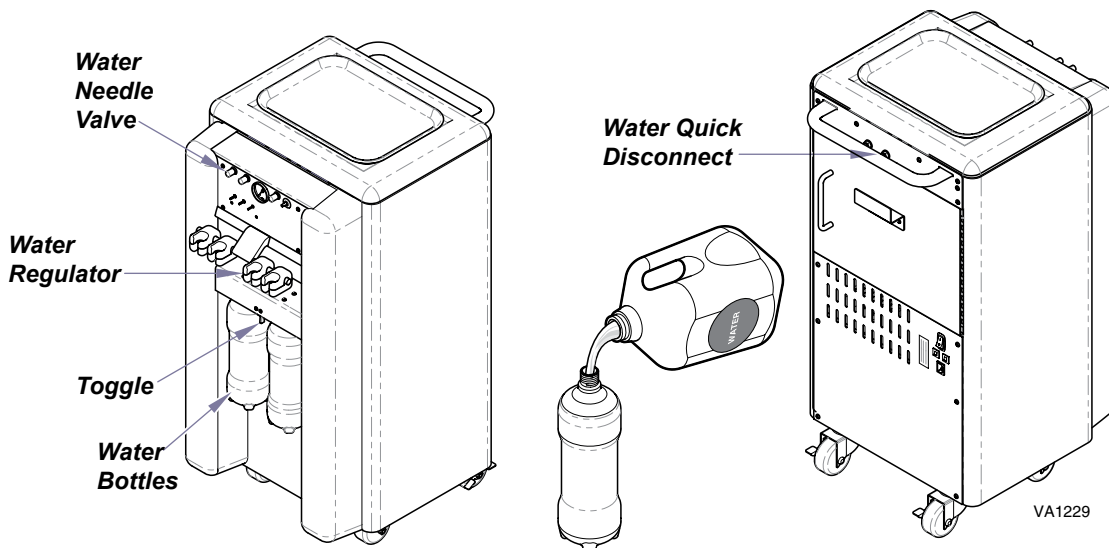
Failure to properly seal water bottles can lead to patient injury.

Water Bottles

The unit has two 1 liter bottles. The water bottles supply water to the handpieces, air/water syringe and a quick disconnect on the back of the cart. Air flows through the water regulator and the toggle switch to the water bottles. The water system is regulated to 40 PSI. Use the water needle valve to adjust water flow for each instrument. The toggle switch between the water bottles that controls which bottle will supply the water. When the toggle is in the center neither water bottle is pressurized. Move the toggle toward the left to pressurize the left bottle and depressurize the right bottle. Toggle to the right to pressurize the right bottle and depressurize the left bottle.

Water Quick Disconnect

The water quick disconnect has a blue ring. You can attach accessories to the cart that will utilize the water from the cart. You may need to add a regulator if attaching equipment that can not handle 40 PSI.

To fill water bottles...

- A) Move toggle to center.
- B) Remove water bottle and fill. (Distilled Water Is Recommended)
- C) Replace water bottle and move toggle toward filled water bottle.

Note: Water bottles should be checked in between each patient and filled if necessary.

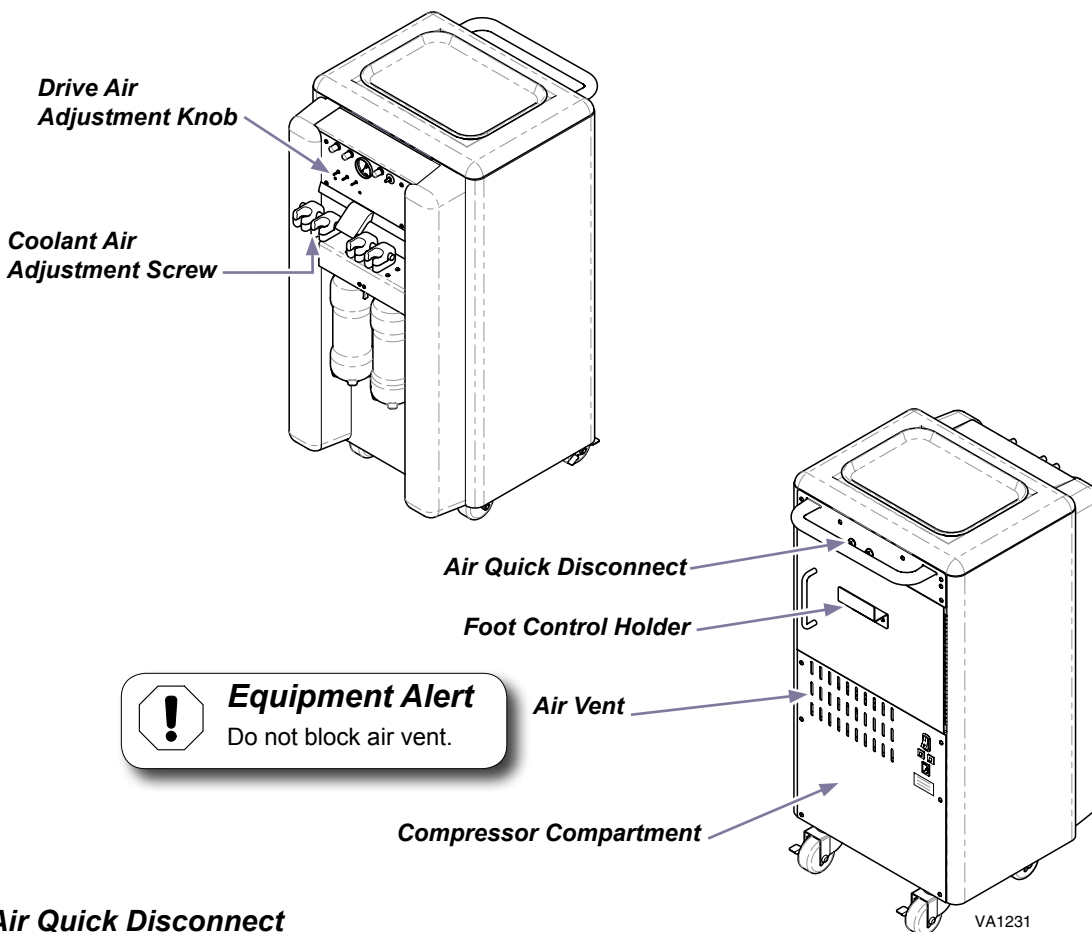
Operation

Air System

Compressor

The air system is a 1/3 HP compressor with a 2 liter air tank and regulator. It supplies air for the handpieces, air/water syringe, and a quick disconnect on the back of the cart. The compressor will cycle between 70 to 100 PSI. The regulator is factory set at 65 PSI.

Air Quick Disconnect

The air quick disconnect has a red ring. You can attach accessories to the cart that will utilize the air from the cart. You may need to add a regulator if attaching equipment that can not handle 100 PSI.

Operation

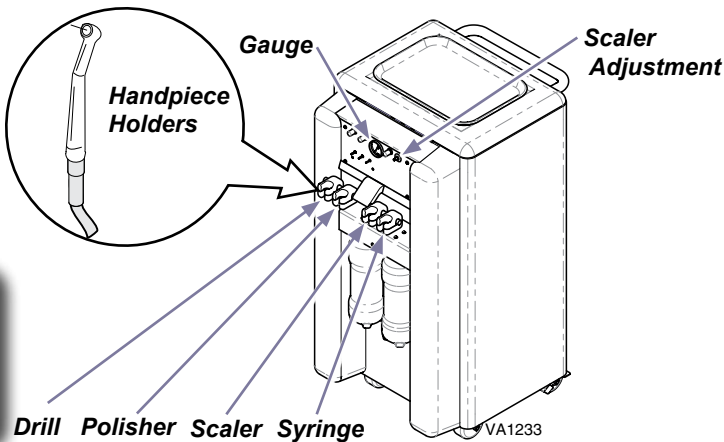
Delivery System

Handpieces

The delivery system controls up to four handpieces. The handpiece pressure is controlled by handpiece drive air adjustment valves. The gauge on front of the delivery system indicates the handpiece pressure. Handpiece pressure should not exceed 60 PSI under normal operation. The scaler tool (optional) has an adjustment knob for the intensity level.


Caution
 Handpieces and Syringe Tip should be changed or disinfected between each patient.

Equipment Alert
 Refer to handpiece manufacturer for suggested PSI settings. Only check one handpiece at a time.

Air/Water Syringe

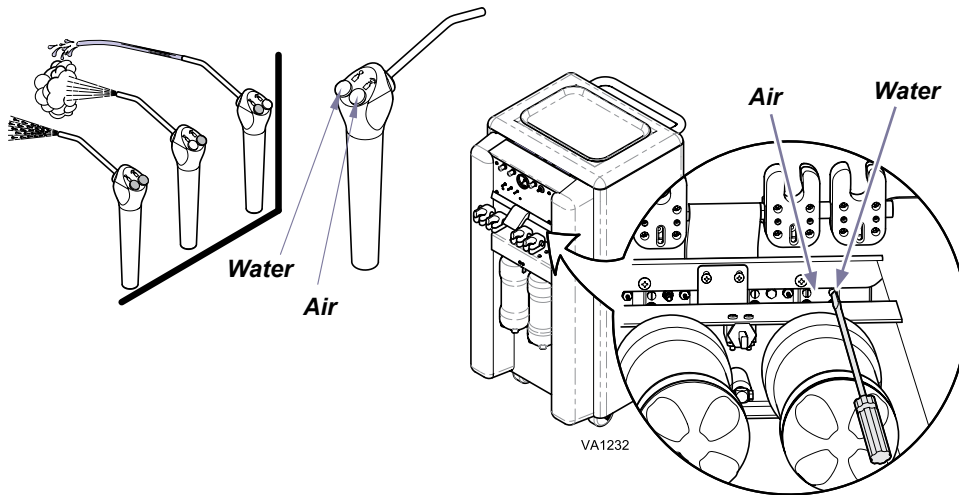
There are air and water buttons on top of the syringe. When both buttons are depressed you will get a mist of both water and air. Air and water adjustments are located under the syringe holder. Insert flat head screwdriver into adjustment screw(s). Counter-Clockwise to increase flow and Clockwise to decrease flow. Turn screw until desired air/water flow is coming through the instrument.

Operation

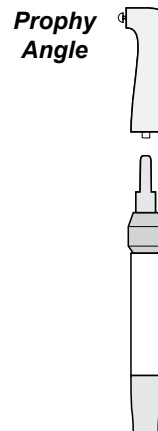Prophy Angle Replacement

To Change Prophy Angle...

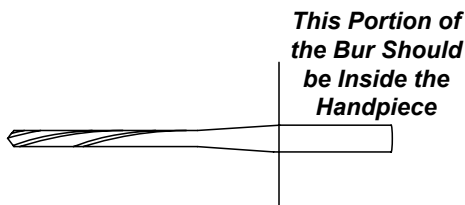
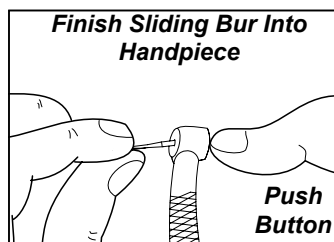
- A) Rotate chuck to the right to unlock.
- B) Install Prophy angle, turn chuck release ring left to tighten.

Burr Replacement

To Change Bur in High Speed Handpiece...

- A) Push button on back of handpiece and pull the bur out.
- B) Push bur into handpiece. (The shank will only go in half-way)
- C) Then push button on back of handpiece and push the bur in until shank is all the way in.

Note: Friction grip burs (FG Style) are designed so that the entire straight portion of the shaft should be inside the chuck of the handpiece. You should only see the tapered portion of the bur if installed properly. New bur recommended for each procedure.


VA1546

Syringe Tip Replacement

To Change Syringe Tip...

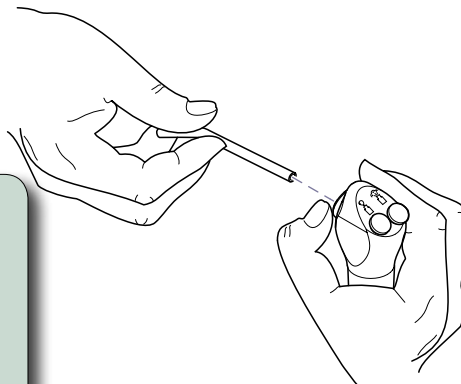- A) Press down on large collar.
- B) When a soft “click is felt pull tip straight out.
- C) Hold collar down and insert new tip. Be sure to press it all the way in, then release collar.
- D) Test the installation by giving a firm pull on tip, to assure it is inserted all the way and locked in place.

VA18071

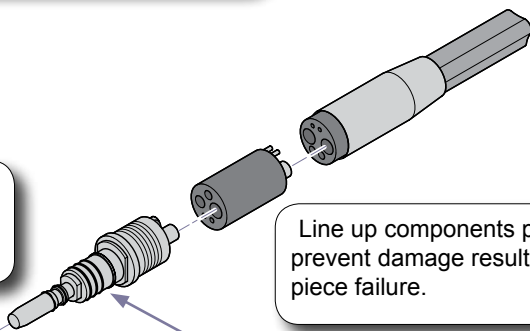
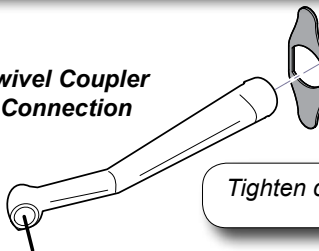
Handpiece Installation

Install handpieces.

Note

Not all units will be equipped with lamp module.


Swivel Coupler Connection


Line up components properly to prevent damage resulting in hand piece failure.

Lightly spray the exposed o-rings with one step conditioner.

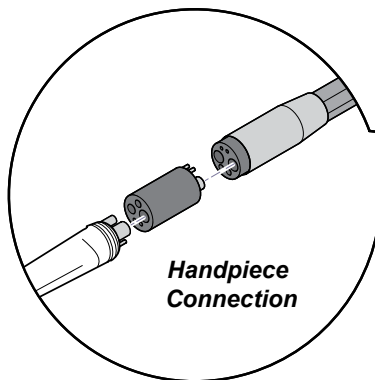Tighten coupler firmly with wrench.

Handpiece Connection

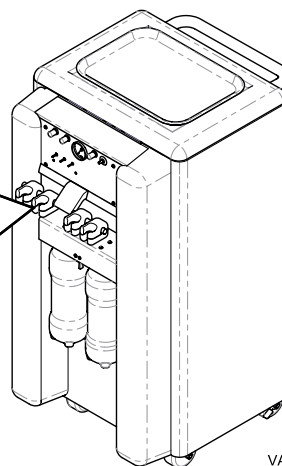
VA1553

Drive Air Adjustment & Locking Casters

To adjust drive air...

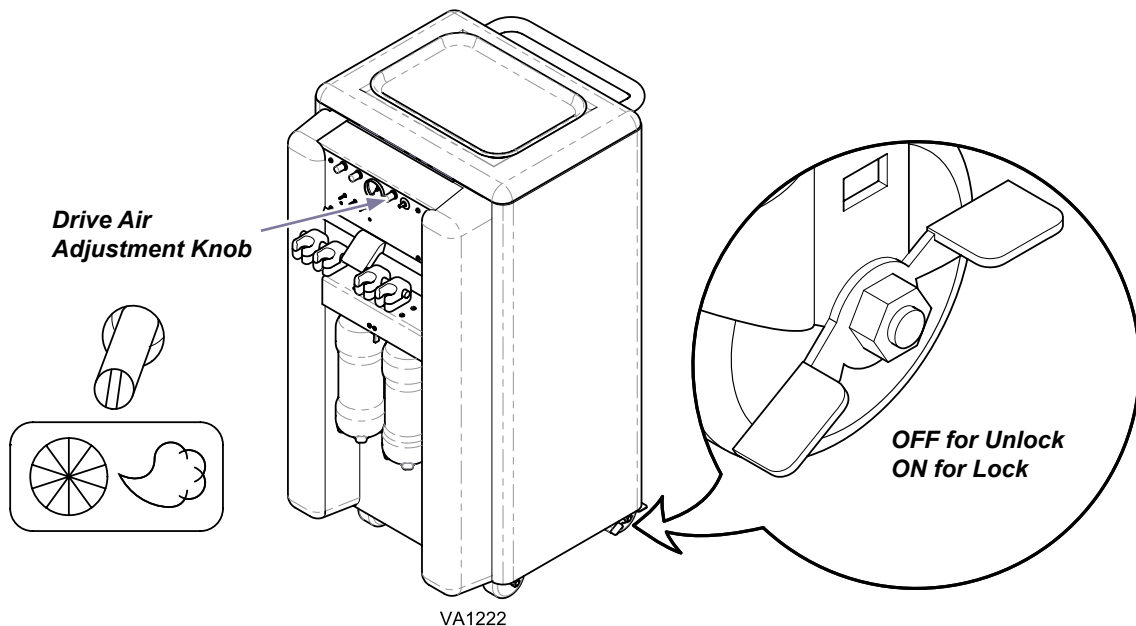Insert flat head screwdriver into knob for Drive Air Adjustment (for desired handpiece). Turn knob until proper air pressure displays on the pressure gauge.

Operation

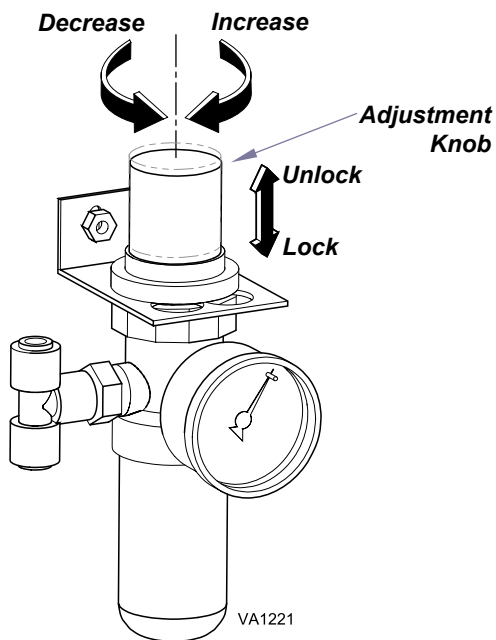
Air Pressure Regulator Adjustment

EQUIPMENT ALERT
Do not adjust regulator over
65 PSI.

To Adjust Regulator...

- A) Pull adjustment knob upward to unlock.
- B) Rotate adjustment knob clockwise to increase pressure, counterclockwise to decrease pressure.
- C) Push adjustment knob downward to lock.



Water Flow Adjustment

To adjust water flow...

Rotate Water Needle Valve (for desired handpiece) until desired flow is achieved.

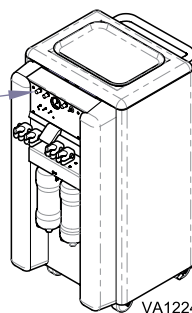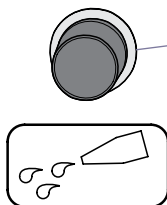Water Needle Valve


Operation

Coolant Air Adjustment

To adjust coolant air...

Insert flat head screwdriver into knob for Coolant Air Adjustment (for desired handpiece). Counter-Clockwise to increase flow and Clockwise to decrease flow. Turn screw until desired air pressure is coming through the instrument.

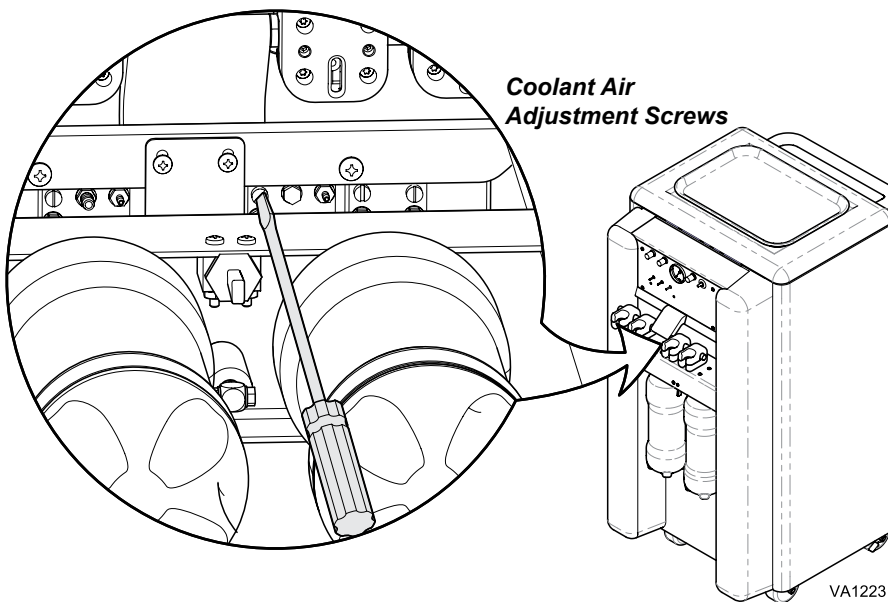Note: Drive air adjustment screws are located below tool holders for each instrument.


Cleaning

- Use cleaners that are appropriate for the situation, such as warm water and mild detergents, or a 10% solution of bleach with 90% water.
- Clean the exterior surfaces with a clean, lint free cloth, dampened with disinfectant. Do not drip any liquid into open vents, plugs or connectors. Dry the surfaces with a clean cloth or paper towel.
- After cleaning, visually inspect the product for deterioration of covers. Do not use the delivery system if excessive discoloration, cracking, or other signs of wear are noticeable (See Calling for Service instructions).

Note:

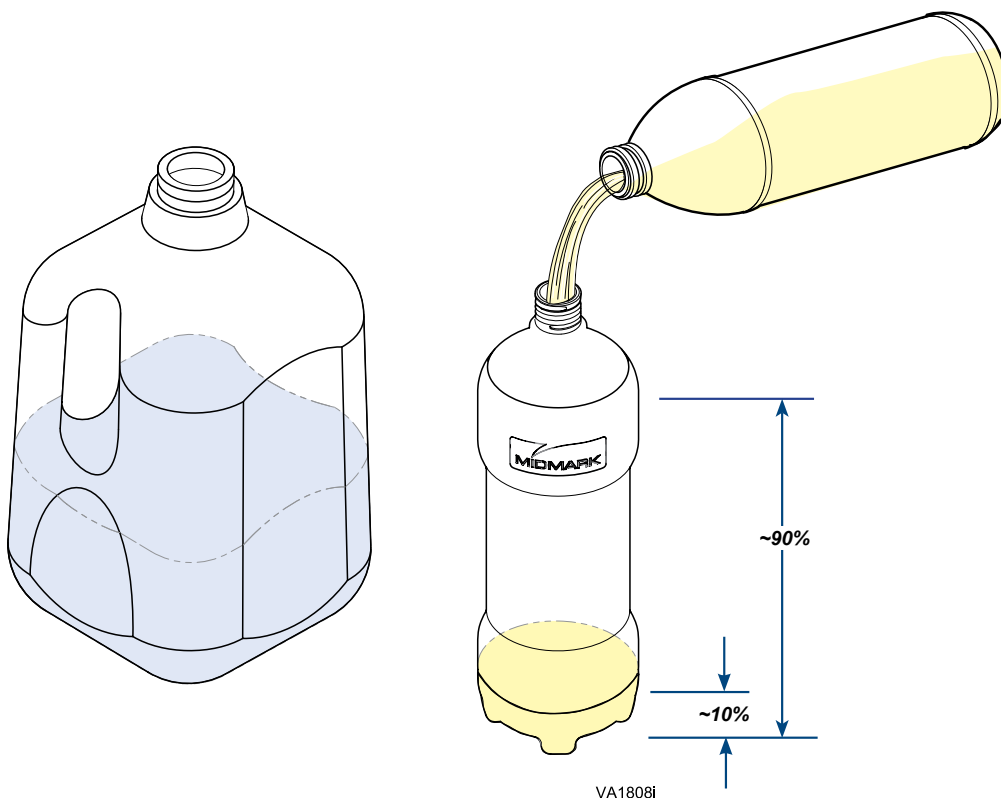Use only 5.25% Bleach Solution. Do not use bleach with scents, color guards, or other additives. Mix fresh every day.

Cleaning *continued...*

Note

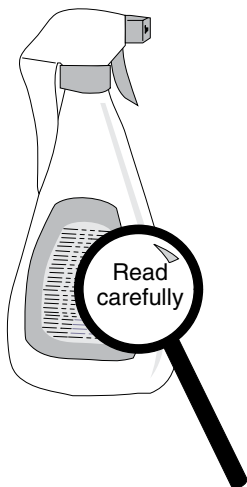
Every veterinary practice setting is different and no one disinfectant is the best choice for every facility. There are several organizations educated in disinfection procedures. Refer to web sites of accredited sources that can assist veterinary personnel in choosing what is best for their practice. Listed below are a few of the sites.

Organization for Safety & Asepsis Procedures


<http://www.osap.org>

Dept. of Health & Human Resources Centers for Disease Control & Prevention (CDC)

<http://www.cdc.gov>

DA1481001

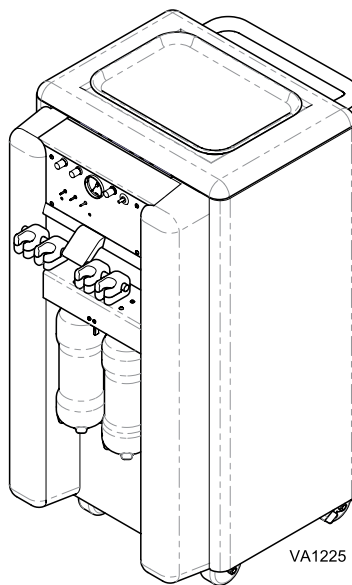VA1225

When using disinfectants...

- Carefully read the product label and directions for use.
- Do not exceed the dilution rate.

Equipment Alert

Refer to handpiece manufacturer for suggested cleaning procedures.

Maintenance

At the beginning of each work day...

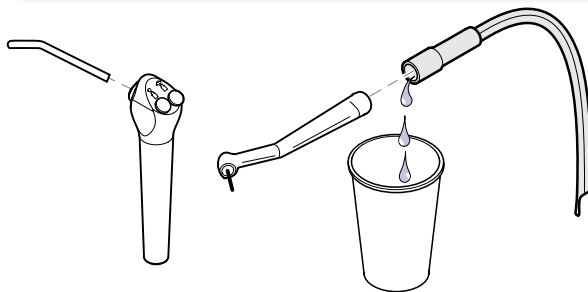
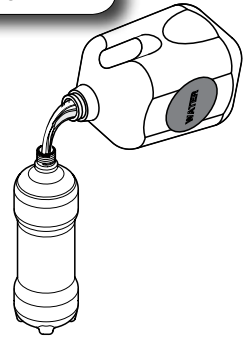
- Fill water bottles with fresh distilled water.
- Flush handpieces.

Note



Distilled water is not mandatory. However, water must meet requirements for drinking.

To Flush Delivery System..

- Remove all handpieces that use water.
- Open water needle valve.
- Hold end of tubing over a container and depress foot control.
- Allow water to run 20 to 30 seconds.
- Install newly disinfected handpieces and syringe tip.
- Refill water bottle.

Water Needle Valve

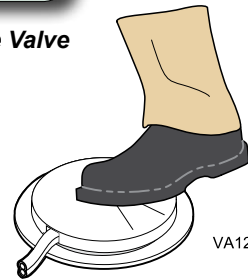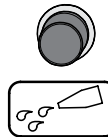VA1226

Between each patient...

- Remove disposable tips, instruments, etc..
- Flush delivery system.
- Disinfect handpieces and scaler per manufacturers instructions.
- Disinfect syringe tip.
- Fill water bottles.

Equipment Alert

HANDPIECE STERILIZATION

Between each procedure complete manufacture's instructions for "Infection Control / Sterilization found in the documentation included with the handpieces. Failure to do this will cause premature failure of handpieces.

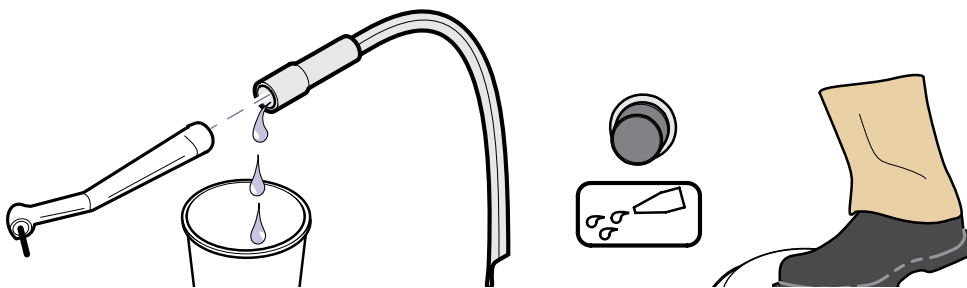
Equipment Alert

AUTOCLAVABLE SYRINGE TIP STERILIZATION

The autoclavable syringe tips supplied with the delivery system must be sterilized prior to use with each patient, including initial use. Be sure to thoroughly rinse and clean syringe tips prior to sterilization, any debris may reduce the effectiveness of the sterilization. Recommended sterilization process is steam autoclave. Recommended parameters are 125°C (250°F) and 106 kPa (15 PSI) for 40 minutes at temperature and pressure.

Maintenance *continued...*

EQUIPMENT ALERT

Do not use colored/scented bleach, or concentrations greater than recommended. Do not leave bleach solution in delivery system longer than 30 minutes, it could damage the delivery system.

To Bleach Flush Delivery System..

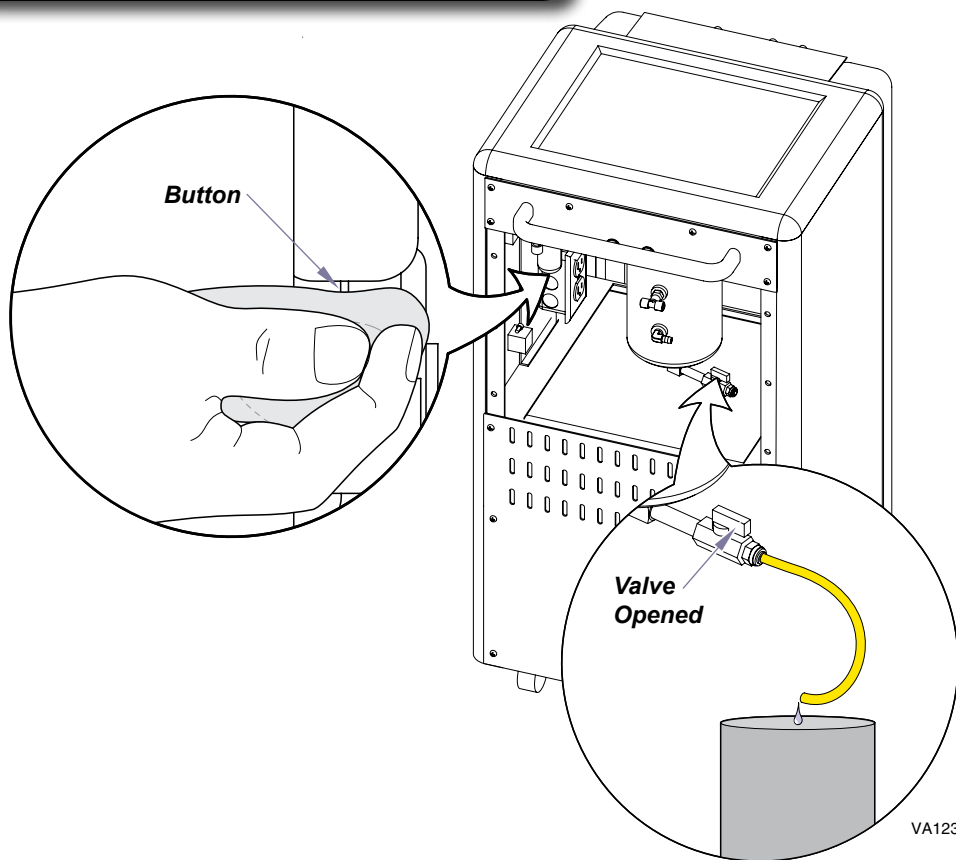
- A) Remove all handpieces that use water.
- B) Remove water bottles and fill with 9 parts water and 1 part bleach.
- C) Open water needle valve.
- D) Hold end of tubing over a container and depress foot control.
- E) Allow water to run 20 to 30 seconds.
- F) Leave bleach solution in delivery system for 10 - 20 minutes, then remove / empty / rinse water bottles.
- G) Fill water bottles with fresh water and repeat steps C thru E until bleach odor is gone.
- H) Install newly disinfected handpieces and syringe tip.
- I) Empty water bottles.

Maintenance *continued...*

To Drain Air Regulator...

Place a cloth under the regulator and depress the button until there is no water coming out.


To Drain Condensation from Air Tank...


- A) Turn power switch off.
- B) Position end of red tube so it drains into a container.
- C) Turn drain valve slowly to open.
- D) When no water is coming out, slowly close valve.

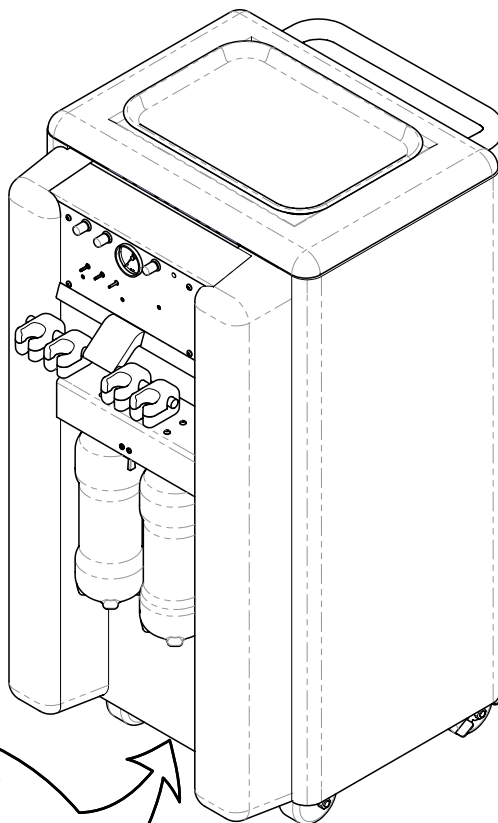Maintenance *continued...*

VA1241

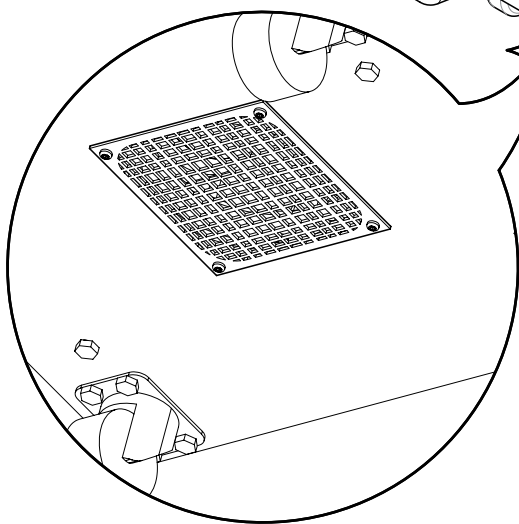Each Month...

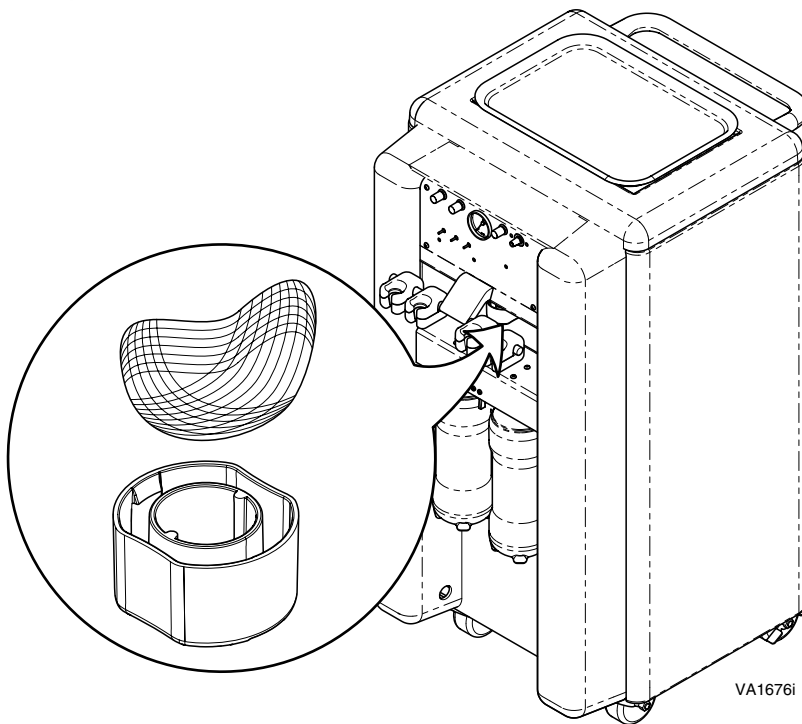
- *Clean screen underneath cart with a rag.*

Maintenance *continued...*

To clean the air/oil separator...

- A) Turn master switch OFF.
- B) Remove container by squeezing sides in while pulling out.
- C) Remove fluid and saturated gauze.
- D) Clean container and mounting cap.
- E) Install clean gauze and reinstall the container.

Calling For Service

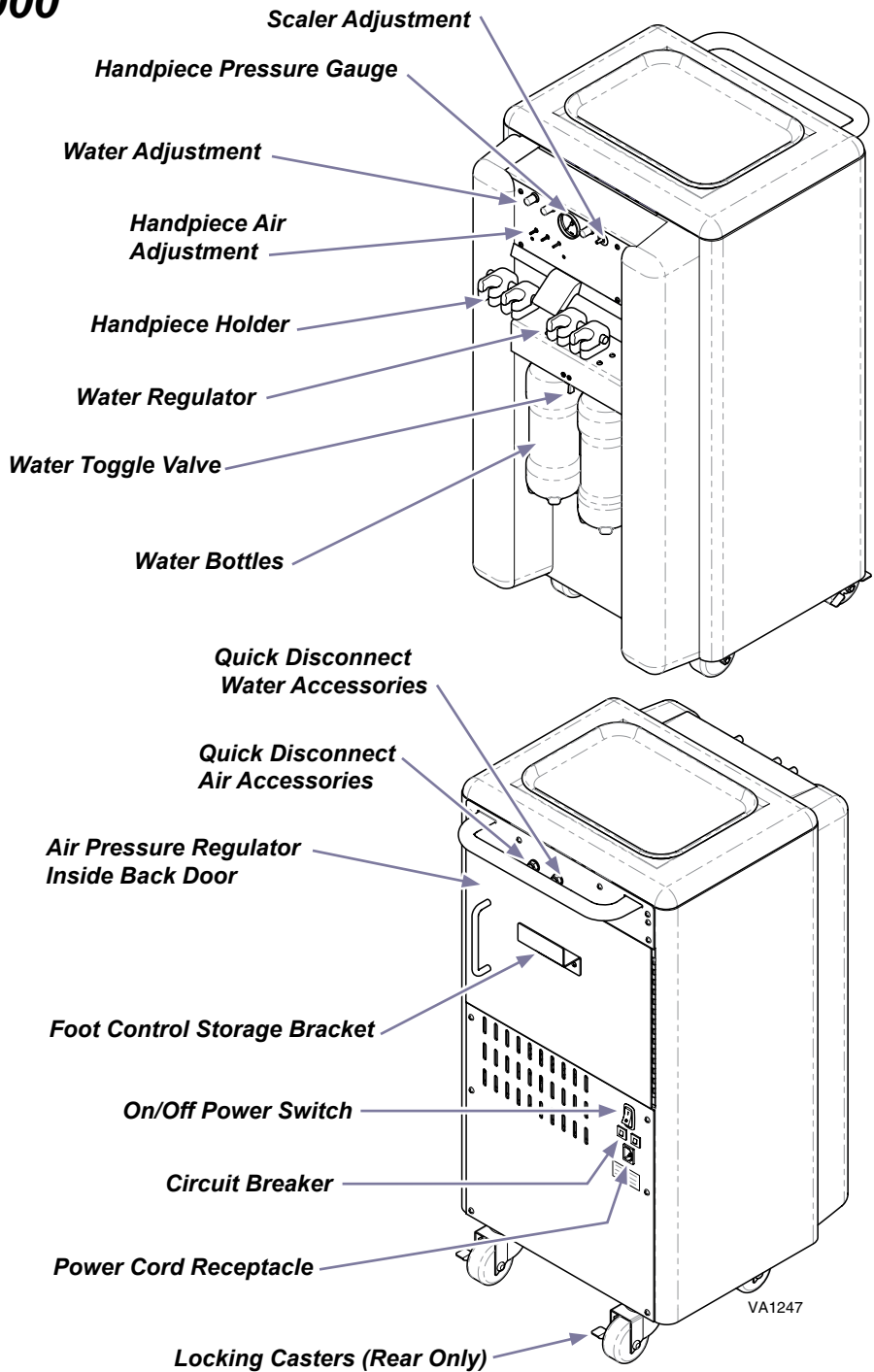
Direct all service inquiries to your authorized Midmark dealer. When calling for service, you must provide the following information:

- Model / serial number
- Date of purchase
- Symptom(s) of malfunction

Service and Parts Manual is available at www.midmark.com.

VetPro® 1000

Specifications

Cart	
Width	23 inches (58.42 cm)
Length	18 1/2 inches (46.99 cm)
Height	36 1/2 inches (92.71 cm)
Weight	130 lbs (58.97 kg)
Shipping Weight	150 lbs (68.04 kg)
Top of Cart Weight Restriction	25 lbs (11.33 kg)
Compressor	
Motor	Oil Free 1/3 HP
Working Pressure	70-100 PSI
Flow Rate	2.5 scfm @ 40 PSI
Sound Level (full load)	40 dB
Air Tank	1 Gallon
Water Module	
Reservoirs	Two - 1 Liter - Toggled
Pressurized System	30-40 PSI
Aseptic	Non-Retracting HPC Coolant
Handpiece Flush System	Manual
Air and Water Connections	External Quick Connects
Other	
Duty Cycle	Continuous Operation Note: Exception, Units with LED Scaler, 10 min. ON, 5 min. OFF
Classifications	Class 1, Type B Applied Part, except as noted for optional accessories, Ordinary Equipment [IPXO]
Optional Accessories:	
Acteon Satelec Scaler	Type BF Applied Part
High Speed Handpiece with and without out Fiber Optics	Type B Applied Part
Low Speed Handpiece	Type B Applied Part
Air Water Syringe	Type B Applied Part

Specifications *continued...*

Electrical (See Regulatory Compliance Chart - Next Page)	
Fuse (Scaler Transformer)	T0.8AL, 250V, 3AG
Circuit Breakers (2) 115VAC	6A 250V, Push to Reset
Circuit Breakers (2) 250VAC	3A 250V, Push to Reset
Power Cord (115VAC)	15A/125VAC x 10 ft. Hospital Grade
Power Cord (230VAC)	10A/250VAC x 2.5 meters
Handpiece	
Connection Type	Midwest 4-Hole Type Handpiece

WARNING

Equipment not suitable for use in the presence of a flammable anesthetic mixture.

WARNING

Do not modify this equipment without authorization of the manufacturer.

WARNING

*To avoid the risk of electric shock, this equipment must **ONLY** be connected to supply mains with protective earth.*

WARNING

The Acteon Newtron® LED scaler complies with current electromagnetic compatibility standards. However, it must not be used by or on those with cardiac stimulators or other active implants unless adequate electromagnetic interference shielding is present. Check with a cardiologist to verify the EMI shielding capabilities of the implant. See enclosed Newtron® LED manual for further advisories.

Model Identification / Compliance Chart

Model	Description	Complies To:				Electrical Ratings:		
		UL 60601-1, Ed. 1	CAN / CSA 22.2, #601.1-M90	IEC 60601-1, Ed. 2	CE	VAC +/- 10%	Amps	Cycles (Hz)
8000-001	VetPro 1000 w/ Scaler	•	•	•	•	115	6	60
8000-002	VetPro 1000 w/o Scaler	•	•	•	•	115	6	60
8000-003	VetPro 1000 w/ Scaler and Fiber Optics	•	•	•	•	115	6	60
8000-004	VetPro 1000 w/o Scaler and Fiber Optics	•	•	•	•	115	6	60
8000-005	VetPro 1000 w/ Scaler, Export				•	230	3	50
8000-006	VetPro 1000 w/ LED Scaler and Fiber Optics. Export				•	230	3	50
8000-006S	VetPro 1000 w/ LED Scaler and Swivel Fiber Optics. Export				•	230	3	50
8000-007	VetPro 1000 w/ LED Scaler and 2 Position Fiber Optics	•	•	•	•	115	6	60
8000-007S	VetPro 1000 w/ LED Scaler and 2 Position Swivel Fiber Optics	•	•	•	•	115	6	60
8000-008	VetPro 1000 w/ Scaler and Fiber Optics	•	•	•	•	115	6	60
8000-008S	VetPro 1000 w/ Scaler and Swivel Fiber Optics	•	•	•	•	115	6	60

Warranty

SCOPE OF WARRANTY Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will repair or replace components of the animal health products manufactured by Midmark (except for products and components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. The sole remedy under this limited warranty is the repair or replacement, at Midmark's option, of the applicable products or components. This limited warranty shall only apply to defects that: (i) are reported to Midmark within the applicable warranty period; and (ii) are determined to exist upon examination by Midmark. This limited warranty extends only to the original retail purchaser of a product, and is not transferable or assignable.

APPLICABLE WARRANTY PERIOD The applicable warranty period, measured from the date of delivery to the original user, shall be as follows: **The VetPro® 1000 unit is warranted against defect in material and workmanship for a period of two years from the time of delivery.**

PRODUCTS are warranted against defect in material and workmanship for the period indicated below from the time of delivery:

- a) 2 years for all products and components (except for the periods in b and c).
- b) Accessories not manufactured by Midmark are excluded (i.e. Fiber optic systems, Scalers, Handpieces, etc.).
- c) Replacement parts and accessories carry a 90 day warranty

OBTAINING WARRANTY SERVICE Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at www.midmark.com, by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380, or by phone at: 1-800-MIDMARK.

It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within thirty (30) days after purchase in order to obtain the benefits of this limited warranty.

EXCLUSIONS This limited warranty does not cover and Midmark shall not be liable for the following:

- (1) defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident (including animal acts of any kind), freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) matching of color, grain or texture except to commercially acceptable standards;
- (3) changes in color caused by natural or artificial light;
- (4) products which are not installed, used and properly cleaned and maintained as required in the installation and operation manuals for the applicable product (imaging product must be installed by a certified Midmark installer);
- (5) products considered to be of a consumable nature;
- (6) accessories or parts not manufactured by Midmark;
- (7) specially manufactured products;
- (8) charges by anyone (including Midmark's authorized dealers) for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (9) costs and expenses of routine maintenance and cleaning;
- (10) all sinks, faucets and plumbing accessories;
- (11) representations and warranties made by any person or entity other than Midmark; and
- (12) with respect to software that is a product or a component thereof, that the software will be error free, can be used without problems or interruptions, or will be free from vulnerability to intrusion or attack by viruses or other methods.

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER Midmark's only obligation under this limited warranty is the repair or replacement of defective parts. Midmark shall not be liable for and hereby disclaims any direct, special, indirect, incidental, exemplary or consequential damages or delays including, but not limited to, damages for loss of profits or income, loss of use, downtime, cover, and employee or independent contractor wages, payments and benefits. This disclaimer shall survive any failure or asserted failure of the essential purpose of this limited warranty or its remedies specified herein.

NO AUTHORIZATION No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with Midmark products.

WARRANTY DISCLAIMER: THIS LIMITED WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

STATUTE OF LIMITATIONS No action may be brought against Midmark for breach of this limited warranty, an implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the warranty period. In the event multiple warranty periods exist with respect to a product, the ninety (90) day period provided for herein shall begin to run from expiration of the warranty period for the component to which the claim relates.

SEVERABILITY In the event any provision of this limited warranty is determined to be invalid or otherwise unenforceable: (i) the provision shall be enforced in a manner that closest holds to its intent, while at the same time curing the invalidity or unenforceability, and (ii) the balance of this limited warranty, being severable, shall not be affected in the event the provision that is invalid or unenforceable cannot be enforced in any respect.

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